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ChemSurf CDS47A

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ChemSurf CDS47A

3D Chemical Indicator

For disinfection processes control of environments and surfaces by hydrogen peroxide aerosol.

Usage

3D chemical indicator for monitoring tests of disinfection of environments and surfaces by Hydrogen Peroxide aerosol.

Conditions:

- H₂O₂ solution concentration ≥ 6%,
- Injected quantity: 5 ml solution of H₂O₂ / m³ room.
- Exposure time ≥ 2 hours.
- Temperature: 10-30 °C.
- RH: 30-80%.

Applicable Regulation

Designed under Quality Management Systems standards ISO 13485:2016/NS-EN ISO 13485:2016.

Characteristics

3D Chemical Indicator with pyramidal shape. Size: 10.5 cm side and 9.0 cm high (approximate). Initial color: bordeaux. Final color: green (should change homogeneously on all exposed faces).

In its base it has a graph to mark the position of both the equipment and the ChemSurf indicator in the room to be disinfected. It also has a record chart to complete the date, operator and result.

Supports are included that make it possible to suspend the indicator.

100 % Toxic Heavy Metals free.

To interpret results visually, use the reference found in the indicator.

Environmental conditions during manufacture

Temperature = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep out of direct light in the original packing.

Transportation conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

Shelf-life

5 years.

Packing

100 units, 5 seals and 5 supports per packing. Documentation table.

Packing information: product code and description, process for intended use, presentation, manufacturer information and storage conditions.

Labelling

On product: product code and description, process for intended use, color change, batch number, expiration date, record chart and datamatrix code.

On product's packing: product code and description, process for intended use, color change, batch number, manufacture and expiration date, barcode and datamatrix code.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to Steam, Dry Heat or any sterilization processes other than H₂O₂.

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